



Requirements:

- Dome Lights per zone
- Pull Cord stations per zone
- Patient station with the call cord per zone
- Alarm Live panel with multi access
- Push Notifications to the end points
- Audio alerts with the custom message per zone over PA system
- Wireless handsets NCS dedicated with the call alerting
- Phone System with the NCS separate layer integration
- Automated recurring reports in Excel and PDF format
- Video Surveillance system
- Access Control multi-zone support
- Remote access and monitoring options



Solution provided:


- TekTone Dome Lights with the replaceable color LED bulbs
- Pull Cord stations and Button Stations with the call cord plug
- Event Management Server with the Live panel, reports, evidence and analytics
- Live Panels per each zone
- Mobile handsets and desk phones for staff and management
- PA system, amplifier and speakers for annunciation and audio alert messaging
- Phone PBX server with the multi-tenant support
- Video Surveillance Server with the video analytics
- Access Control system with the evidence control
- Network, workstation, remote management and Wi-Fi



Equipment provided





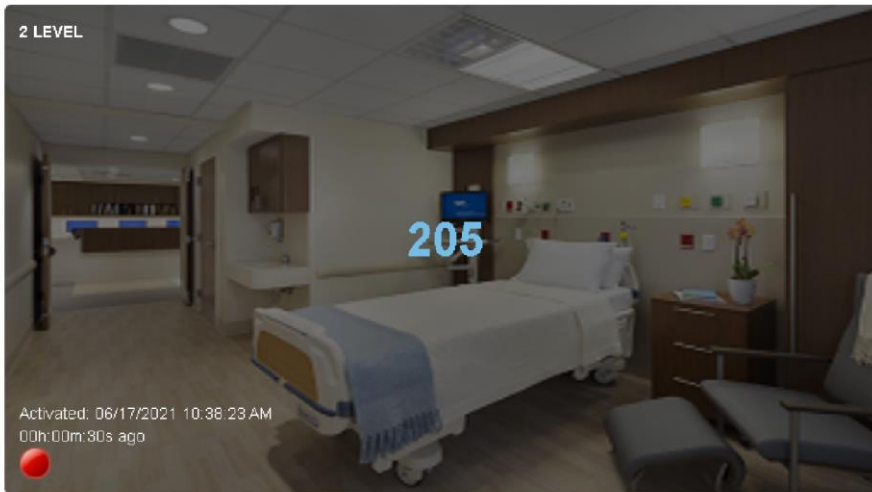

- HELP ALARM BUTTON - LEVEL 1 ROOM 136
 Activated: 06/16/2021 04:06:19 PM
 Canceled: 06/16/2021 04:08:30 PM
 Response: 00h:02m:11s 17h:52m:25s ago
- PULL CORD - LEVEL 1 ROOM 106
 Activated: 06/16/2021 04:04:59 PM
 Canceled: 06/16/2021 04:07:27 PM
 Response: 00h:02m:28s 17h:53m:29s ago
- HELP ALARM BUTTON - LEVEL 1 ROOM TABLE-004
 Activated: 06/16/2021 12:31:26 PM
 Canceled: 06/16/2021 12:31:37 PM
 Response: 00h:00m:11s 21h:29m:18s ago
- PULL CORD - LEVEL 1 ROOM 105
 Activated: 06/07/2021 11:23:44 AM
 Canceled: 06/07/2021 11:24:00 AM
 Response: 00h:00m:16s 238h:36m:55s ago
- PULL CORD - LEVEL 1 ROOM 112
 Activated: 06/07/2021 11:22:42 AM
 Canceled: 06/07/2021 11:22:59 AM
 Response: 00h:00m:17s 238h:37m:57s ago
- HELP ALARM BUTTON - LEVEL 1 ROOM 136
 Activated: 06/07/2021 11:13:03 AM
 Canceled: 06/07/2021 11:13:24 AM
 Response: 00h:00m:21s 238h:47m:31s ago
- PULL CORD - LEVEL 1 ROOM 106
 Activated: 06/07/2021 11:11:57 AM
 Canceled: 06/07/2021 11:12:09 AM
 Response: 00h:00m:12s 238h:48m:46s ago
- PULL CORD - LEVEL 1 ROOM 112
 Activated: 06/04/2021 10:48:55 AM
 Canceled: 06/04/2021 10:52:21 AM

CONTROLLED CARE

- Dashboard
- Live Panel
- Device Record Test
- Analytics
- Response
- Response Summary
- Activity
- CDR Calls
- Nurse Calls
- Analytics Template
- Staff
- Beacon Log
- Recurring Report
- Application Config
- Admin Config
- User / Role Admin
- Tools

Rooms Select All Rooms and Beds

Date:

Show Chart

Show Top:

Response Time Greater Than:

Friday, June 11, 2021 - Thursday, June 17, 2021

Response Time Sum (daily average)
 Number of Calls

Control Button	Floor #	Department	Room #	Bed #	Total Calls	Response Time
Pull Cord	1		106		1	00h:02m:28s
Alarm Button	1		136		1	00h:02m:11s
Alarm Button	1		Table-004		1	00h:00m:11s

Page 1 of 1

CONTROLLED CARE Copyright © 2018 Master Communicator by ControlledCare.com. All rights reserved.
Version 1.10.1.0

CONTROLLED CARE

Create New

Alegria Bronx NY | Select Department | Select Floor | Enter Search String | Search | Upload rooms from file

Facility	Floor #	Department	Room #	Description	Beds	
Alegria Bronx NY	1		105	Patient Toilet - Infusion Treatment Area		
Alegria Bronx NY	1		106	Patient Toilet - Infusion Treatment Area		
Alegria Bronx NY	1		111	Patient Toilet - Ultrasound Room		
Alegria Bronx NY	1		112	Patient Toilet - Treatment / Exam Room 6		
Alegria Bronx NY	1		136	Patient Toilet - Cast Room		
Alegria Bronx NY	1		137	Patient Toilet - Cast Room		
Alegria Bronx NY	1		150	Patient Toilet - Waiting Area		
Alegria Bronx NY	1		154	Patient Toilet - Waiting Room		
Alegria Bronx NY	1		180	Patient Toilet - PED Waiting Room		
Alegria Bronx NY	1		181	Patient Toilet - Infant Exam Room 1		
Alegria Bronx NY	1		Table-001	INFUSION TREATMENT AREA		
Alegria Bronx NY	1		Table-002	INFUSION TREATMENT AREA		
Alegria Bronx NY	1		Table-003	INFUSION TREATMENT AREA		
Alegria Bronx NY	1		Table-004	INFUSION TREATMENT AREA		
Alegria Bronx NY	1		Table-005	INFUSION TREATMENT AREA		
Alegria Bronx NY	1		Table-006	INFUSION TREATMENT AREA		
Alegria Bronx NY	1		Table-007	INFUSION TREATMENT AREA		

Call History

1 | 2 | 3 | 4 | 5 | >

10 | Caller-ID | Start |

End |

<input type="checkbox"/>	Start	From	To	Agent	Trunk	Duration
6/16/2021						
<input type="checkbox"/>	4:08:19 PM	"ROOM 136" < sip:room-136@10.45.40.109 >	NCS Group (888)	20		00:01
<input type="checkbox"/>	4:06:59 PM	"ROOM 106" < sip:room-106@10.45.40.109 >	NCS Group (888)	60		00:14
<input type="checkbox"/>	4:07:19 PM	"ROOM 136" < sip:room-136@10.45.40.109 >	NCS Group (888)	40		00:01
<input type="checkbox"/>	4:06:19 PM	"ROOM 136" < sip:room-136@10.45.40.109 >	NCS Group (888)			
<input type="checkbox"/>	4:05:59 PM	"ROOM 106" < sip:room-106@10.45.40.109 >	NCS Group (888)			
<input type="checkbox"/>	4:04:59 PM	"ROOM 106" < sip:room-106@10.45.40.109 >	NCS Group (888)			
<input type="checkbox"/>	12:31:26 PM	"TABLE 04" < sip:table-04@10.45.40.109 >	NCS Group (888)			
6/8/2021						
<input type="checkbox"/>	11:59:36 AM	40 NCS (40)	718-642-2727 (718-642-2727)			
<input type="checkbox"/>	11:59:19 AM	40 NCS (40)	718-642-2727 (718-642-2727)			

- [Upload Extra.csv](#)
- [Download Extra.csv](#)
- [Upload Settings](#)
- [Download Settings](#)
- [Load from Global](#)
- [Download Global](#)
- [Reset to defaults](#)
- [Reboot](#)

LV Log

```

2021-06-16 13:28:25,639 [INFO] BLE Socket Server started at 127.0.0.1:8686
2021-06-16 13:28:26,025 [INFO] Reinit GPIOs...
2021-06-16 13:40:07,761 [INFO] Reinit GPIOs complete!
2021-06-16 13:40:07,847 [INFO] Initialization done!
2021-06-16 15:04:55,531 [INFO] Input changed: E21A5 Name: PULL Mode: In01 Change: 0->1
2021-06-16 15:04:55,547 [INFO] Activate trigger: 14
2021-06-16 15:04:55,570 [DEBUG] Check pulse: Elapsed: 4294967295 Duration: 1000
2021-06-16 15:04:57,049 [DEBUG] Send POST request(1 of 3) to http://10.45.40.109/authcode/? :
{"BaseName":"","lv014","EventType":"Alarm","Sensorid":"E21A5","CtrlType":"PULL","Fid":"","Floor":"","Description":"VANTACT","Room":""}
2021-06-16 15:04:59,283 [DEBUG] Empty response (OK)
2021-06-16 15:06:18,295 [INFO] Input changed: E20A7 Name: INFUS Mode: In01 Change: 0->1
2021-06-16 15:06:18,298 [INFO] Activate trigger: 17
2021-06-16 15:06:18,302 [DEBUG] Check pulse: Elapsed: 4294967295 Duration: 1000
2021-06-16 15:06:19,316 [DEBUG] Send POST request(1 of 3) to http://10.45.40.109/authcode/? :
{"BaseName":"","lv017","EventType":"Alarm","Sensorid":"E20A7","CtrlType":"INFUS","Fid":"","Floor":"","Description":"VANTACT","Room":""}
2021-06-16 15:06:19,625 [DEBUG] Empty response (OK)
2021-06-16 15:07:25,390 [INFO] Input changed: E21A5 Name: PULL Mode: In01 Change: 1->0
2021-06-16 15:07:25,400 [INFO] Deactivate trigger: 14
2021-06-16 15:07:26,478 [DEBUG] Send POST request(1 of 3) to http://10.45.40.109/authcode/? :
{"BaseName":"","lv014","EventType":"Normal","Sensorid":"E21A5","CtrlType":"PULL","Fid":"","Floor":"","Description":"VANTACT","Room":""}
2021-06-16 15:07:27,262 [DEBUG] Empty response (OK)
2021-06-16 15:08:29,127 [INFO] Input changed: E20A7 Name: INFUS Mode: In01 Change: 1->0
2021-06-16 15:08:29,130 [INFO] Deactivate trigger: 17
2021-06-16 15:08:30,241 [DEBUG] Send POST request(1 of 3) to http://10.45.40.109/authcode/? :
{"BaseName":"","lv017","EventType":"Normal","Sensorid":"E20A7","CtrlType":"INFUS","Fid":"","Floor":"","Description":"VANTACT","Room":""}
2021-06-16 15:08:30,809 [DEBUG] Empty response (OK)
    
```

- System
- EMS
- GPIO In/out**
- Alarm types
- GPIO groups
- RFID
- BLE devices
- Alarm triggers
- Scheduler
- Audio files
- URLs
- Administration
- Camera
- SIP Accounts
- SIP Params
- SIP Log

On board
MCP #1 on20
MCP #2 on21
MCP #3 on22
MCP #4 on23

E23A0 Name	OUTPUT-01	Description	Relay01	<input type="checkbox"/> DISABLED	<input checked="" type="checkbox"/> OUT(1)	<input type="checkbox"/> OUT(0)	<input type="checkbox"/> IN(0-1)	<input type="checkbox"/> IN(1-0)
E23A1 Name	OUTPUT-02	Description	Room 181	<input type="checkbox"/> DISABLED	<input checked="" type="checkbox"/> OUT(1)	<input type="checkbox"/> OUT(0)	<input type="checkbox"/> IN(0-1)	<input type="checkbox"/> IN(1-0)
E23A2 Name	OUTPUT-03	Description	Table 001	<input type="checkbox"/> DISABLED	<input checked="" type="checkbox"/> OUT(1)	<input type="checkbox"/> OUT(0)	<input type="checkbox"/> IN(0-1)	<input type="checkbox"/> IN(1-0)
E23A3 Name	OUTPUT-04	Description	Table 002	<input type="checkbox"/> DISABLED	<input checked="" type="checkbox"/> OUT(1)	<input type="checkbox"/> OUT(0)	<input type="checkbox"/> IN(0-1)	<input type="checkbox"/> IN(1-0)
E23A4 Name	OUTPUT-05	Description	Table 003	<input type="checkbox"/> DISABLED	<input checked="" type="checkbox"/> OUT(1)	<input type="checkbox"/> OUT(0)	<input type="checkbox"/> IN(0-1)	<input type="checkbox"/> IN(1-0)
E23A5 Name	OUTPUT-06	Description	Table 004	<input type="checkbox"/> DISABLED	<input checked="" type="checkbox"/> OUT(1)	<input type="checkbox"/> OUT(0)	<input type="checkbox"/> IN(0-1)	<input type="checkbox"/> IN(1-0)
E23A6 Name	OUTPUT-08	Description	Table 005	<input type="checkbox"/> DISABLED	<input checked="" type="checkbox"/> OUT(1)	<input type="checkbox"/> OUT(0)	<input type="checkbox"/> IN(0-1)	<input type="checkbox"/> IN(1-0)
E23A7 Name	OUTPUT-07	Description	Table 006	<input type="checkbox"/> DISABLED	<input checked="" type="checkbox"/> OUT(1)	<input type="checkbox"/> OUT(0)	<input type="checkbox"/> IN(0-1)	<input type="checkbox"/> IN(1-0)
E23B0 Name	OUTPUT-16	Description	Room 154	<input type="checkbox"/> DISABLED	<input checked="" type="checkbox"/> OUT(1)	<input type="checkbox"/> OUT(0)	<input type="checkbox"/> IN(0-1)	<input type="checkbox"/> IN(1-0)
E23B1 Name	OUTPUT-15	Description	Room 150	<input type="checkbox"/> DISABLED	<input checked="" type="checkbox"/> OUT(1)	<input type="checkbox"/> OUT(0)	<input type="checkbox"/> IN(0-1)	<input type="checkbox"/> IN(1-0)
E23B2 Name	OUTPUT-14	Description	Table 012	<input type="checkbox"/> DISABLED	<input checked="" type="checkbox"/> OUT(1)	<input type="checkbox"/> OUT(0)	<input type="checkbox"/> IN(0-1)	<input type="checkbox"/> IN(1-0)
E23B3 Name	OUTPUT-13	Description	Table 011	<input type="checkbox"/> DISABLED	<input checked="" type="checkbox"/> OUT(1)	<input type="checkbox"/> OUT(0)	<input type="checkbox"/> IN(0-1)	<input type="checkbox"/> IN(1-0)
E23B4 Name	OUTPUT-12	Description	Table 009	<input type="checkbox"/> DISABLED	<input checked="" type="checkbox"/> OUT(1)	<input type="checkbox"/> OUT(0)	<input type="checkbox"/> IN(0-1)	<input type="checkbox"/> IN(1-0)
E23B5 Name	OUTPUT-11	Description		<input type="checkbox"/> DISABLED	<input checked="" type="checkbox"/> OUT(1)	<input type="checkbox"/> OUT(0)	<input type="checkbox"/> IN(0-1)	<input type="checkbox"/> IN(1-0)
E23B6 Name	OUTPUT-10	Description		<input type="checkbox"/> DISABLED	<input checked="" type="checkbox"/> OUT(1)	<input type="checkbox"/> OUT(0)	<input type="checkbox"/> IN(0-1)	<input type="checkbox"/> IN(1-0)

IV Control v1.4 © 2020
Powered by CONTROLLEDCARE